



DENTIRO® Zero

Alcohol-free, ready-to-use surface disinfectant for the rapid disinfection and cleaning of non-invasive medical devices.

- ▶ **Kills 101 microorganisms**
- ▶ **Short exposure time**
- ▶ **Compatible with more than 250 materials including artificial leather**
- ▶ **Alcohol-free**

DENTIRO® Zero is an alcohol-free, ready-to-use solution for the fast disinfection and cleaning of non-invasive medical devices with sensitive surfaces such as dental chairs. DENTIRO® Zero is particularly suitable for the disinfection of alcohol-sensitive materials such as artificial leather, Plexiglas®, and PVC.

Scope of Application



- ▶ Alcohol-free, ready-to-use surface disinfectant for the quick disinfection and cleaning of non-invasive medical devices.

Instructions for Use



For your protection, wear disposable gloves and safety goggles. Completely moisten and wipe the entire surface of the medical device. Leave disinfectant to act and adhere to contact time. Before first time use, test material compatibility by applying the disinfectant to a small inconspicuous area. A list of compatible and incompatible materials is available on our website. If you are uncertain as to the material composition of the medical device, contact the manufacturer before use. Wipe medical devices, which come into contact with skin, once a day with a water-soaked single-use cloth to prevent the build-up of biocides.

Effective Spectrum / Exposure Times



Bactericidal	60 sec
Antibiotic-resistant bacteria	60 sec
Tuberculocidal	60 min
Mycobactericidal	60 min
Yeasticidal	60 sec
Antibiotic-resistant yeasts	60 sec
Enveloped viruses	15 sec
Bovine viral diarrhea virus (BVDV)	15 sec
Coronavirus	15 sec
Hepatitis B virus	15 sec
Hepatitis C virus	15 sec
Herpes simplex virus	15 sec
Human immunodeficiency virus (HIV)	15 sec
Influenza A virus	15 sec
Vaccinia virus	15 sec
Virucidal	60 min
Adenovirus	30 min
Norovirus	60 min
Poliovirus	60 min
Rotavirus	30 sec

Complies with EN 13624, EN 13727, EN 14348, EN 14476, EN 14563 and EN 16615.





Compatibility

Acrylic glass
Polyethylene
Polypropylene
Polyvinyl chloride
Silicone rubber
Steel

Compatible
Compatible
Compatible
Compatible
Compatible
Compatible

No signal word according to Regulation (EC) 1272/2008.

H412 Harmful to aquatic life with long lasting effects. P273 Avoid release to the environment. P305 + P351 + P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. P501 Dispose of contents/container in accordance with local and national regulations. For more information, please refer to Safety Data Sheet available on our website.

Composition in 100 g

0.4 g benzalkonium chloride, 0.4 g N-alkyl aminopropyl glycine, auxiliaries and water.

Physical Properties

Physical state: Clear, non-viscous liquid
Colour: Colourless
Scent: Apple
pH-value: 8.0 - 8.5
Shelf Life: 48 months (unopened container)

Delivery Forms

250 ml bottle, 1 litre bottle with flip top cap, 5 litre canister

Your Local Distributor

O|C|C Switzerland

Oro Clean Chemie AG
Allmendstrasse 21
8320 Fehraltorf
Switzerland
Telephone: +41 (0)44 226 44 44
info@oroclean.com
www.oroclean.com
<https://www.facebook.com/Oroclean>